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PRODUCT INFORMATION UMBILICAL HERNIA CORSET


English

Description: Umbilical hernia corset provides elastic compression support for umbilical hernia and abdominal hernia. The corset features a silicone pad with a breathable cotton cover.

Sizing:

SIZING	BELLY SIZING	ART.NR.
S	75 - 85 cm	PO 31423 S
M	85 - 95 cm	PO 31423 M
L	95 - 105 cm	PO 31423 L
XL	105-115 cm	PO 31423 XL
XXL	115 - 125 cm	PO 31423 XXL

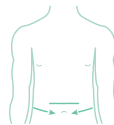Indications: Umbilical hernia (umbilical hernia), Scar hernia (cicatricalis hernia), Abdominal wall hernia (abdominal hernia)

Manual:

1. Place the silicone pad on the navel.
2. Keep the silicone pad positioned on the navel and pull the band tight.
3. Close the Velcro closure with the desired compression for sufficient support

Important information and warnings:

This medical product is intended for use by one individual patient. The product should be used in accordance with your doctor's advice and instructions. The product should be installed by a trained professional the first time.

In cases where the product is recommended by the doctor to be used at night, it must be ensured that it does not interfere with blood circulation. The product should be detached in case of numbness and removed if necessary. Consult your doctor in case of persistent complaints.

The product should not be exposed to excessive heat or flame. Contact with oily or acidic substances or lotions containing pomades should be avoided.

This product must not be used outside its scope or combined with any other product. It should only be used according to the original indications.

Daily use and duration of application should be determined by the physician. Excessive and prolonged use can cause muscle weakness. Always consult your doctor for the correct wearing schedule.

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