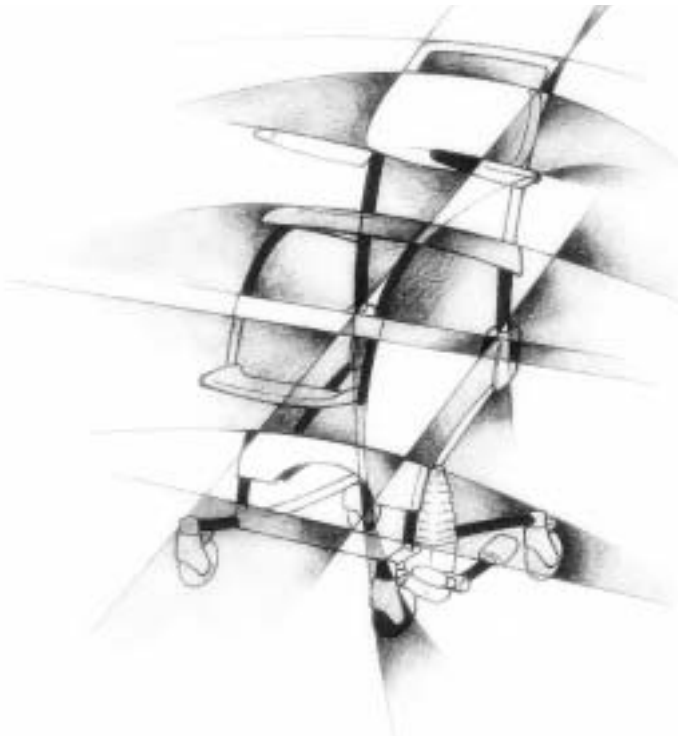
# Flexo

**Gebruiksaanwijzing  
Instructions for use  
Notice d'utilisation  
Bedienungsanleitung**

*D E S I G N E D   F O R   C O M F O R T*

|                                                               |                   |           |
|---------------------------------------------------------------|-------------------|-----------|
| <b>Flexo gebruiksaanwijzing Nederlands .....</b>              | <b>Nederlands</b> | <b>1</b>  |
| <b>Flexo instructions for use English .....</b>               | <b>English</b>    | <b>13</b> |
| <b>Notice d'utilisation de la chaise Flexo Français .....</b> | <b>Francais</b>   | <b>27</b> |
| <b>Flexo bedienungsanleitung Deutsch .....</b>                | <b>Deutsch</b>    | <b>41</b> |

# inhoud

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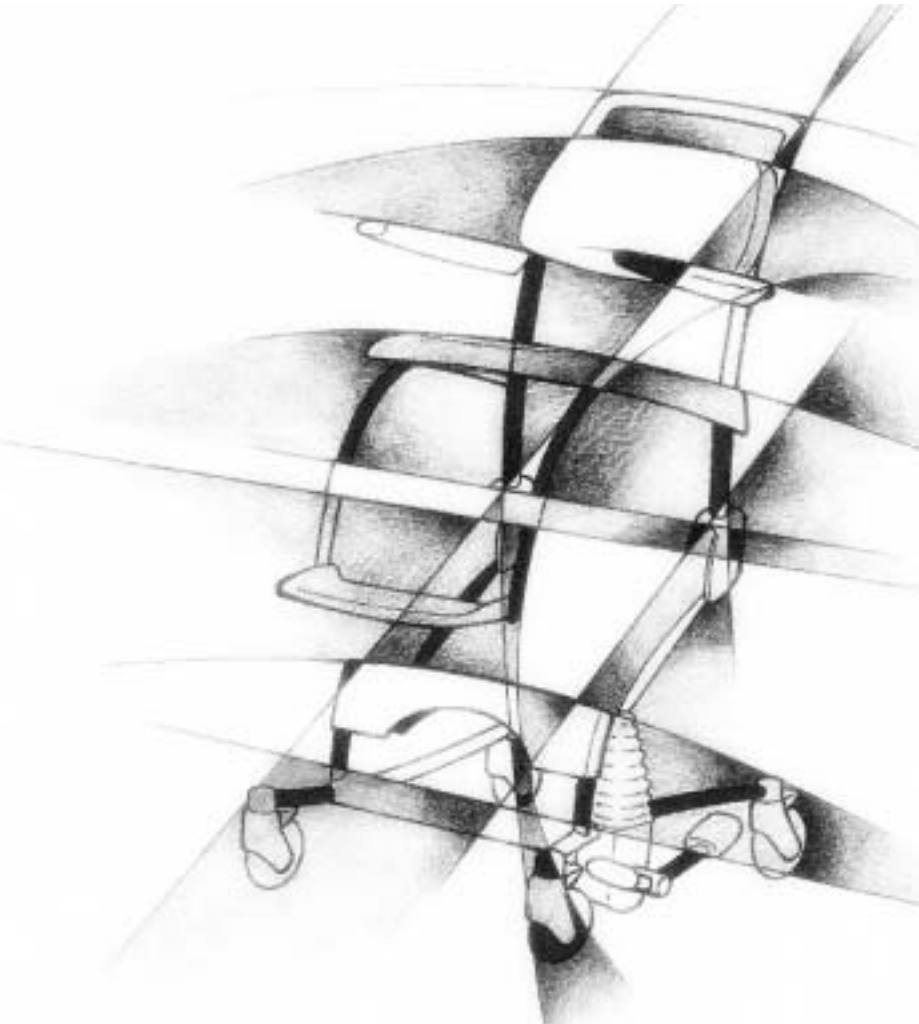
## Introduction

The high/low *Flexo* shower and toilet chair is an aid that allows people to be nursed and washed in an ergonomically sound manner. It offers maximum comfort to both the user and the nurse. The Flexo can also be used as a toilet seat, with the use of a bedpan, as well as in combination with a toilet.

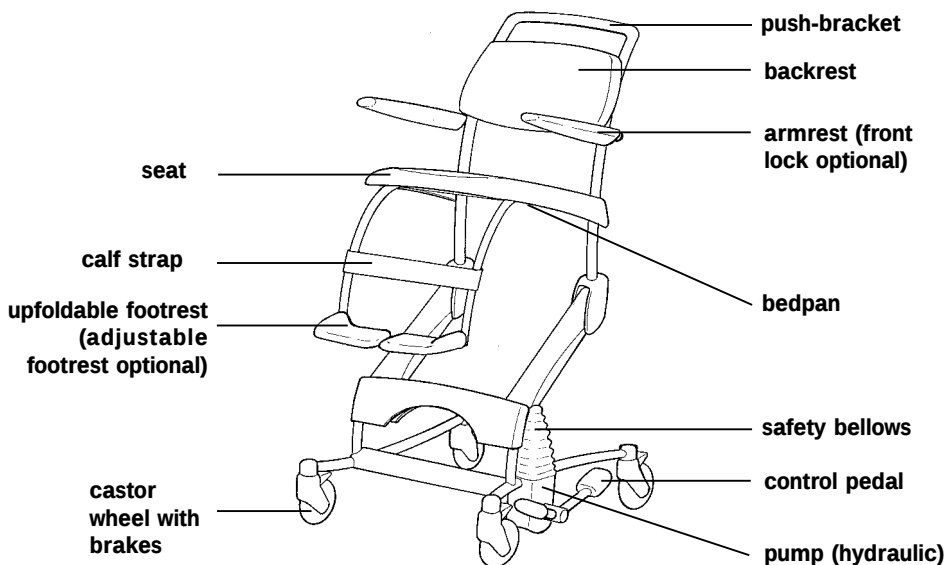
The Flexo consists of a base unit and a seat unit. The seat offers the comfort and the correct support of a normal chair. The seat height from the base can be adjusted hydraulically, making the Flexo a multi-functional chair. The standard Flexo comes with upfoldable footrests and armrests. In addition, various accessory features are available as options, such as a front lock and a adjustable footrest.

The flexo is a product that is supplied with the CE mark. This mark is in conformity with the Directive for Medical Equipment

# 93/42 EEC of June 14th, 1993

## Component names

## Your Flexo

Date of first use:

User

name :

address :

postal code :

city :

Date of expiry of term

guarentee :

Serial number:

Year of manufacture:

Manufacturer:

**INVACARE INTERNATIONAL**

Les Roches

37230 Fondettes

FRANCE

## **Safety, general**

- Ensure that the brakes of the Flexo are always locked during transfers
- Before the user is raised, always lock the safety belt first
- Only ride and manoeuvre the chair in its lowered position
- Do not raise or lower the seat when riding the chair
- If possible, always cross obstacles with the chair in reversed position
- In case of any malfunctioning please contact your dealer

## **Brakes**

- Lock the brake of the Flexo by stepping down on the extended green lip on the wheel
- Unlock the brakes by pressing down the vertical lever of the wheel

## Sitting down / Standing up

### With upfoldable footrests (standard)

- The nurse sets the seat height to fit the user
- The nurse folds up both footrests
- The nurse hangs the -open- safety belt behind the frame
- The user stands up straight and holds on to a fixed support, e.g. a bed or a bracket
- The nurse rides the Flexo from behind towards the user, until the calf-strap touches the user
- When the user needs help sitting down, the nurse will lock the brakes first
- The user lowers himself and sits on the Flexo

*Note: For standing up, follow these steps in reverse order*

## Upfoldable footrest

- The nurse folds down the footrests before the Flexo is raised
- The nurse or the user will lift up one leg first, the nurse will then fold down the footrest; then the same procedure is followed for the other leg

## Safety belt

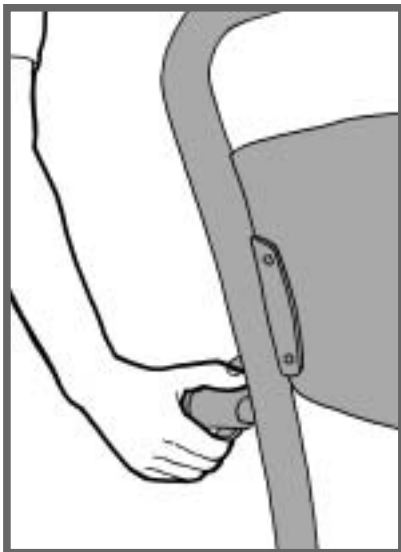
- Before the user sits down on the Flexo, disengage the safety belt and hang it down the back of the seat first
- After the user has sat down, put both halves of the safety belt over the upper legs of the user
- Secure the safety belt by engaging both buckle joints
- Fit the belt by sliding the buckle over the belt

## Armrests

### Standard armrests


- When the armrests are folded up or down, protect the user by holding his or her arm aside

### Armrests with front lock (option)

- Once the user has sat down on the Flexo, fold down the armrests with front lock first, before turning them in front of the user
- The armrests with front lock are locked in horizontal position
- Before this front lock can be folded up and turned away, the safety pen near the frame mounting point of the armrests must be pushed in first
- Always protect the user, when the armrests are folded up or down
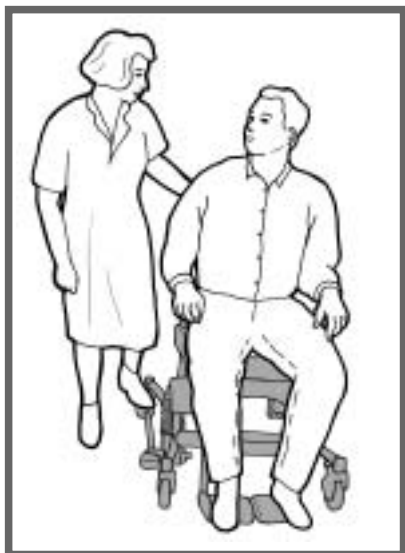
## Raising

- When the user is sitting on the Flexo, the brakes of the Flexo must be locked first, before the seat may be raised
- The seat is raised by quietly and evenly stepping down on the long end of the pedal
- During the stepping movements the upper body the nurse is facing the Flexo lengthways, so the nurse cannot twist his/her upper body in relation to the position of the feet
- Keep in contact with the user during the upward movement

## Lowering

- Lower the seat by pressing down the short end of the pedal
- The seat will lower under the influence of gravity; control the lowering speed by slightly holding down the pedal

*Note: The lowering speed can also be controlled well by lifting up the long end of the pedal with the foot*

## **Using the bedpan**

- To use the Flexo as a toilet seat, the middle part must be removed from the seat before the user sits down
- Using the special bedpan guide, slide the bedpan under the seat from the rear
- Turn the bedpan at the end of the guide surface, so the handle will click around the knob

## Using the toilet function

- First remove the middle part from the seat before the user sits down, when the Flexo is moved around a toilet
- Raise the Flexo to the correct height for the toilet
- Move the Flexo backwards until it is positioned above the toilet
- If required, lower the Flexo to its correct position

*Note: The Flexo does not fit over all toilet types*

## Washing

- Lower the Flexo into the required position for washing the upper body and the hair
- Raise the Flexo into the required position for washing the lower body and the legs
- Because the height of the Flexo is adjustable, the nursing and washing of the user is ergonomically improved for the nurse
- The correct height setting depends on the attendant's length

## Maintenance

- The Flexo must be cleaned with water and normal household cleaning agents after each use; do not use any abrasives or aggressive solutions
- Regularly check the wheels for hairs and dust and, if found, remove them
- Remove any excess soap residue

## Technical data

### Structure

- Frame of coated steel tubing and plate materials
- Stainless steel turning points
- Plastic wheels, O.D. 125 mm, dual blocking system
- PUR (polyurethane) body support
- Drive system: integrated hydraulic unit

### Performance

- Persons weighing up to 135 kg
- Seat height: 50 cm minimum, 105 cm maximum

### Dimensions

- Ground surface: 70 cm wide, 82 cm deep
- Tilt angle seat: Lowest position: 3°  
Highest position: 15°
- Wedge angle: 100°
- Overall weight of the Flexo: 40 kg

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